

1.1. G3 Sterile Nitrile Gloves

1.1.1. Inspect packaging of G3 Sterile Glove wallet for the following

- Confirm you have selected the proper size
- Package integrity is good and package is sealed
- Product has not expired

1.1.2. To Don first layer of sterile gloves aseptically, locate the top of package and open by peeling the sterile glove polyethylene pouch

1.1.3. Remove glove wallet, place on a clean surface and unfold by pinching the sides of the wallet to open (do not touch the exterior surface of the gloves)

1.1.4. Apply the first glove to the non-dominant hand by sliding hand palm into the glove (thumb facing outward). Bend the thumb toward the center of the palm and slide thumb into the glove while pulling on the inner portion of the cuff with the dominant hand. Leave the cuff rolled up

1.1.5. Apply the second glove to the dominant hand by using the four gloved fingers to cup under the cuff of the second glove. Slide the ungloved palm (thumb facing outward) into the glove. Bend the thumb toward the center of the palm and slide into the glove while pulling upward with the fingers of the gloved hand. Completely unfold the cuff

1.1.6. Complete donning the gloves by placing the fingers of the second glove under the cuff of the first glove and unfolding the cuff

1.1.7. Sanitize using 70/30% IPA before proceeding